MYOCONNECTOR

INFORMATION FOR SAFETY AND PERFORMANCE

(EN)

1. PRODUCT DESCRIPTION

MYOCONNECTOR adapters are sterile and strictly disposable devices, intended for use in conjunction with FIAB MYOPACE® epimyocardial pacing wires for temporary cardiac electrostimulation in connection with temporary electrostimulators or threshold analysers, during and after cardiac surgery.

2. INDICATIONS AND USERS

MYOCONNECTOR adapters are indicated for temporary cardiac electrostimulation and sensing of the cardiac signal, after cardiac surgery, in order to prevent post-operative bradycardia and conduction disorders. MYOCONNECTOR adapters must be fixed, connected and operated by experienced healthcare personnel, competent in cardiac surgery procedures.

3. INTENDED PURPOSE

Temporary single- and/or dual-chamber cardiac electrostimulation of adult and paediatric patients in connection with temporary electrostimulators, during and after cardiac surgery.

4. ESSENTIAL PERFORMANCE

Allow for bidirectional transmission of electrical signals (pacing and sensing) between the pulse generator and the heart, also ensuring the best power transfer jointly to excellent mechanical strength and flexibility properties.

5. CONTRAINDICATIONS

- Do not use with pacing wires other than FIAB MYOPACE®.

6. POSSIBLE COMPLICATIONS

For complications related to the use of the product, please refer to the instructions for use of the FIAB MYOPACE® epimyocardial pacing wires.

7. PRECAUTIONS AND WARNINGS

To reduce complications the following precautions need to be taken.

7.1 Sterility

WARNING!

SINGLE-USE DEVICE – DO NOT REUSE – DO NOT RESTERILIZE

**WARNING!**

Single-use product sterilized with Ethylene Oxide. Do not re-use nor re-sterilize. Re-use may result in: loss of the product's initial functional features, alteration of materials, presence of biological residues that may cause cross-infections.

**WARNING!**

Before opening the package, carefully check that it has not been damaged in such a way as to compromise the sterility of the content.

**WARNING!**

Sterility is guaranteed up until the expiration date indicated on the package, only if it has been properly stored and is not damaged; in case of damage or visible defects, do not use the product and return it to FIAB.

7.2 Equipment and accessories

**WARNING!**

MYOCONNECTOR adapters must be used in conjunction with external cardiac pacemakers (CF class) that comply with EN 60601-1 and EN 60601-2-31. Before use, check the instructions for use of the equipment for any limitations or requirements that may preclude the use of the adapters.

**WARNING!**

In order to prevent induced currents from being channelled towards the heart with the risk of inducing life-threatening ventricular arrhythmias, electrically insulate the pins if they are disconnected from the cardiac stimulator (use the provided caps).

7.3 Performance and use

**WARNING!**

The device must be fixed, connected and operated by a physician specialised in cardiac surgery.

**WARNING!**

It is essential to always have a package of MYOCONNECTOR available.

**WARNING!**

As long as the pin is not connected to the temporary stimulator, it must be covered by its cap.

**WARNING!**

Any connection of an extension cable must be carried out before the pins of the implanted pacing wires are in turn connected.

**WARNING!**

The epimyocardial pacing wires to which MYOCONNECTORs are connected form direct current pathways to the patient's heart. Throughout their use, only battery-powered equipment or mains-powered equipment specifically designed for this purpose must be used to ensure protection against fibrillation caused by alternating currents. In addition, mains-powered equipment used in close proximity to the patient must be adequately earthed.

**WARNING!**

The metal pin of the MYOCONNECTOR adapter must be isolated from any current leakage that may arise from mains-powered equipment.

**WARNING!**

The product must be stored in its original package at the environmental conditions (temperature and relative humidity) specified on the label. Putting external heavy weights on the package could damage the product.

**WARNING!**

The product may be contaminated after use, constituting a biological hazard. The device and its parts must be handled according to medical procedure in order to prevent injury or infection.

Rev.	Revision Status / Description	Date
A	First Issue	25/02/2025

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